

Tandem® Microspheres

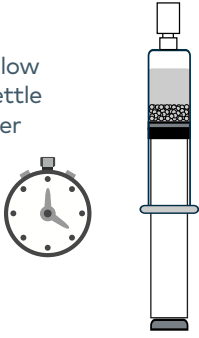
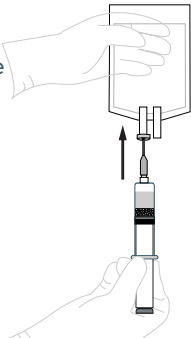
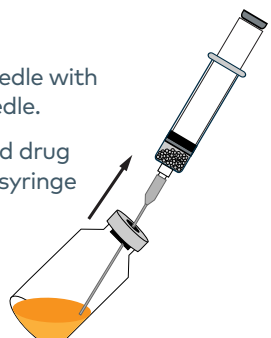
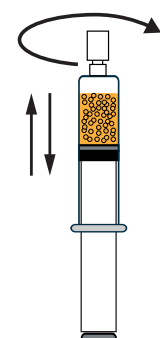
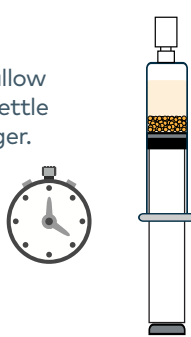
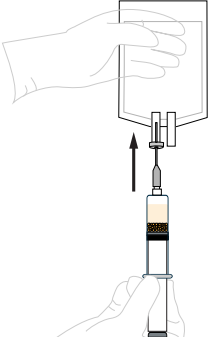
Loading Guide for Irinotecan HCl 20 mg/ml Solution

Materials:

- Tandem Microspheres prefilled syringe
- 5 µm sterile filter needles (2)
- 18–20 gauge sterile needle
- Waste container (e.g., empty 100 ml solution bag)
- Irinotecan hydrochloride (20 mg/ml solution)

Drug loading should be performed under controlled aseptic conditions.

Maximum loading dose:
50 mg irinotecan/ml microspheres

MICROSPHERE PREPARATION	
<p>1</p> <p>Gently swirl and allow microspheres to settle towards the plunger</p> 	<p>2</p> <p>Attach filter needle to the syringe.</p> <p>Insert needle into a waste container and expel excess transport solution. DO NOT compress the microspheres.</p> 
DRUG LOADING	
<p>3</p> <p>Replace filter needle with 18-20 gauge needle.</p> <p>Draw up required drug volume into the syringe and leave an air bubble for optimal mixing.</p> 	<p>4</p> <p>Recap the syringe.</p> <p>Gently mix by inverting and swirling the syringe every 5 minutes. Hard shaking may cause foam.</p> <p>Minimum loading time is 30 minutes.</p> 
STORAGE (OPTIONAL)	SUPERNATANT REMOVAL
<p>After drug loading is complete, if not used immediately, store in the dark at 2-8° C and use within 48 hours.</p> <p>Please refer to the drug manufacturer's package insert for specific drug information.</p>	<p>5</p> <p>Gently swirl and allow microspheres to settle towards the plunger.</p> 
	<p>6</p> <p>Attach filter needle to the syringe.</p> <p>Insert needle into a waste container and expel excess supernatant. DO NOT compress the microspheres.</p> <p>Recap the syringe.</p> 

CAUTION: The law restricts these devices to sale by or on the order of a physician.

Indications, contraindications, warnings, and instructions for use can be found in the product labeling supplied with each device. Vascular embolization is a procedure that has inherent risk and should only be performed by specialized physicians. For more information, please visit www.varian.com/safety Varian Medical Systems as a medical device manufacturer cannot and does not recommend specific treatment approaches.

Information contained herein is for distribution outside the U.S. only. Products and features may not be available in all markets and specifications are subject to change without notice.

© 2020 Varian Medical Systems, Inc. Varian, Varian Medical Systems, Embozene, and Tandem logos are registered trademarks of Varian Medical Systems, Inc. All trademarks are the property of their respective owners.